



# Crescent Barrier Extensions

Vitus Ostomy Crescent Barrier Extensions are curved adhesive strips designed to prevent the edges of barrier from lifting and help keep your pouch secure. The thin, hydrocolloid formulation helps conform to uneven body curves. Crescent Barrier extensions can be used in place of tape and act as a frame, providing an extra layer of protection to minimize leaks. They are highly absorbent and can be used under or over pouch wafer.

- Hydrocolloid formulation
- Provide extra protection
- Water-resistant



Item #	Description	HCPCS	Packaging
513	Vitus Ostomy Crescent Barrier Extensions	A4362	30 ea/tray - 50 tray/cs

## How to use

### Under the Wafer

1. Clean the skin with water and dry, avoid using soap.
2. Remove paper from the crescent barrier extension.
3. Smooth the crescent barrier extension around the stoma. Your natural body temperature will help secure the seal.
4. Secure ostomy wafer over crescent barrier extension.
5. *Optional.* For additional security, place directly around the edges of the wafer. Form a "picture frame" around the border of the wafer, using 1-3 extensions.

### Over the Wafer

1. Clean the skin with water and dry, avoid using soap.
2. Secure ostomy wafer to skin.
3. Remove paper from the crescent barrier extension and place directly around the edges of the wafer. Form a "picture frame" around the border of the wafer, using 1-3 extensions.
4. Smooth the crescent barrier extension around the wafer with your finger. Your natural body temperature will help secure the seal.

Live life *freely*