



Skin Barrier Rings

Vitus Ostomy Conforming Adhesive Seals are ideal for any type of ostomy. Available in three sizes, our skin barrier rings create a tight seal to prevent leaks, even with high liquid output. They are conformable and stretchable without unnecessary mess and residue. A great alternative to barrier paste.


- High absorbency prevents leaks
- Helps maintain healthy skin around stoma
- Paste alternative
- Latex-Free
- Low Cost



Item #	Description	HCPCS	Packaging
520	Vitus Ostomy 2" Skin Barrier Rings - Thin	A4385	20 ea/bx - 20 bx/cs
521	Vitus Ostomy 2" Skin Barrier Rings - Standard	A4385	20 ea/bx - 20 bx/cs
522	Vitus Ostomy 4" Skin Barrier Rings	A4385	10 ea/bx - 20 bx/cs

How to use

1. Clean the skin with water and dry, avoid using soap.
2. Remove paper from the seal.
3. Mold and stretch seal snugly around the stoma or place directly onto ostomy wafer (per your clinician's instructions).
4. Smooth the seal around the stoma or wafer with your finger. Your natural body temperature will help secure the seal.

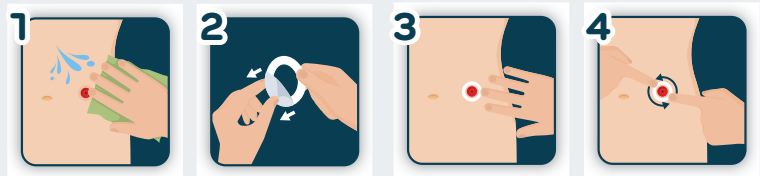

Live life *freely*